



Wave (Z220 with tub transfer base Z228)

Wave

A Bathing & Transfer System

Hygiene and health go together. When we make bathing easier, our clients are healthier. The Wave system has been thoroughly reimagined to improve bathing and showering for the client and for the caregiver.

Responding to requests from therapists and caregivers, we widened the seat to accommodate larger clients. And to protect skin integrity, we added generous padding under soft knit fabric.

In addition to the tub stand and rolling shower stand, we've added an ingenious tub transfer base to completely change the way you transfer and bathe large clients. There's nothing like it.



Find letters of medical necessity
and informative articles at:
www.rifton.com/wave

The Wave bath chair is terrific! In the past, lifting a child into a tub has been a back killer, but the new transfer base eliminates this hazard entirely. The casters allowing lateral movement in the tub are a welcome addition, and the fact that I can easily collapse the base and hang it on the wall is a fabulous space saver. ”

*Linda K. Miller, DPT
Queensbury, New York*

Wave features



Generously sized

The wide seat comfortably accommodates large clients. The seat width on the large size is 44 cm.



One-hand adjustment

Adjust backrest angle with one hand, keeping the other hand on the client for safety and reassurance.



Full leg support

The calf rest provides full support for the lower legs. Adjust it independently of the seat by pushing the gray buttons and rotating up or down. It must be installed at the time of purchase.



Typical bathtub height range

— 46 cm
— 36 cm





Tub stand



Shower stand



Tub transfer base



Highly adjustable
Backrest angle, seat angle and calf rest angle adjust individually.



More seat heights
For an intermediate seat height, put the rear legs up and front legs down. To get the chair low in the tub, put the front and back legs up. Fold the chair completely flat for storage.



Components

Bath chair size and color



Choice of chest strap




Without lateral positioning

The simple chest strap safely positions the user in the Wave bath chair. It attaches with hook and loop where needed along the full height of the back and buckles securely behind.



With lateral positioning

We designed this chest strap with adjustable lateral straps to give additional lateral support, helping users maintain better trunk alignment in the bath chair. The soft fabric laterals will not get in the way during transfers.

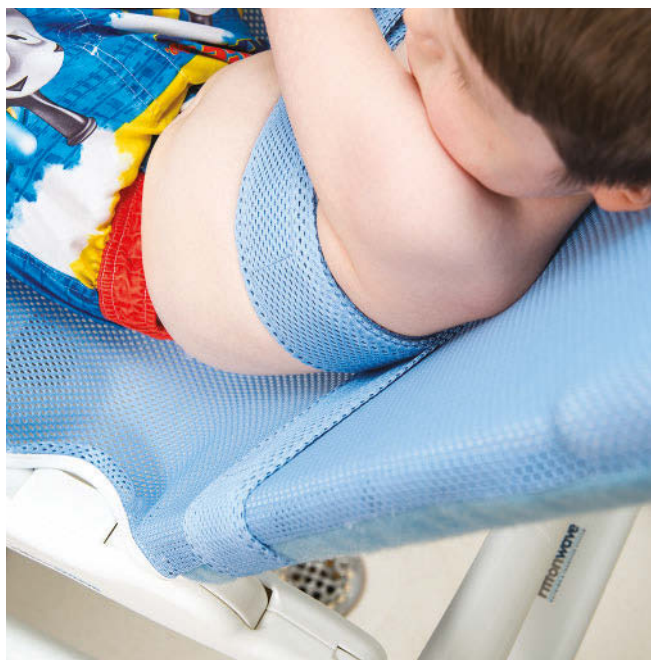



Head blocks

Head blocks provide lateral support and positioning for the user's head.



Leg straps

Use the fully adjustable leg straps to position your client's hips in abduction (or just to provide additional security).



Calf rest

The calf rest supports the lower legs. It is strongly recommended when the bath chair is used on one of the stands.

Note: The calf rest must be chosen at the time of the original purchase. It cannot be added later.



Tub stand

The tub stand raises the bath chair either 14 cm or 29 cm higher in the tub depending on the position of the tub stand legs.



Shower stand

The shower stand lets you roll your client into the shower stall at a convenient height for cleaning. Locking 10 cm casters provide smooth, easy maneuverability.



Components (continued)

Tub transfer base

Bathing equipment for people with disabilities tends to be big and bulky, taking up far too much space in bathrooms shared by the whole family. Using Rifton's tub transfer base, you can perform healthy transfers without crowding your bathroom with permanently mounted equipment. When bath time is over, the base can be folded up and hung compactly on the wall.



Unique two-stage movement



The tub transfer base starts in the position farthest out of the tub. It is then rolled back over the tub, and the brakes are set.



As the upper portion with the bath chair is rotated 90° it moves approximately 30 cm back over the tub in one motion.



The legs of the tub transfer base can be adjusted in increments of 1.5 cm to fit bathtubs up to 61 cm high.



1. With the client secured in the chair, push the transfer base to the back of the tub as far as it will go and lock the wheels.



2. Press either of the gray pivot latches and rotate the bath chair and client over the tub.



3. As the bath chair rotates, it continues to move back over the tub. Gently lift the client's legs to clear the edge of the tub.



4. In the final position the bath chair is over the tub and can be adjusted as desired for showering.


Free up the tub.

Rifton's folding, storable tub transfer base makes it possible for everyone in the family to still use the tub.

When you are finished, remove the base from the tub, fold the legs and hang it on the wall hook provided.



Wave dimensions

User dimensions (cm)	Z210 small	Z220 medium	Z230 large
Height	up to 117	102–142	127–188
 <p>Key user dimension: height Select the appropriate bath chair by the user's overall height. Choose the size that allows for growth.</p> <p>Important: Make sure that seat depth, width and back height are adequate. The user's weight must not exceed the maximum working load.</p>			
	Item dimensions (cm)	Z210 small	Z220 medium
Seat depth	27	33	43
Seat width	38	38	44
Back height	58	67	81
Seat height	5–18	5–18	5–18
Seat height (incl. tub stand)	32–47	32–47	32–47
Seat height (incl. shower stand)	70	70	70
Seat height (incl. tub transfer base) with long leg inserts	53–66 62–75	53–66 62–75	53–66 62–75
Length of mesh area when flat (without calf rest)	89	104	128
Length of mesh area when flat (with calf rest)	108	124	155
Base width	46	46	53
Base length (legs down) with legs flat	61 75	61 75	61 75
Tub stand height (legs flat)	20	20	20
Tub stand length (legs flat)	99	99	99
Tub stand width (legs flat)	44	44	44
Shower stand height	56	56	56
Shower stand width	69	69	69
Shower stand length	86	86	86
Tub transfer base footprint	71 x 76	71 x 76	71 x 76
Max. tub height for transfer base with long leg inserts	51 61	51 61	51 61
Maximum working load (kg)	34	68	113
Back and seat angle adjustments		0°, 15°, 30°, 45°, 60°, 75°	



All three sizes are available in blue and pink.

2022 order form

Use dimension chart to select appropriate size.

Conversion kits

The tool-free conversion kit converts your small chair to medium as your child grows.

Kit without calf rest

Blue Z268
Pink Z269

(Includes seat, back, cover and chest strap)



Kit with calf rest

Blue Z266
Pink Z267

(Includes seat, back, cover and chest strap)



		Z210 small	Z220 medium	Z230 large
* Chair		Z211 <input type="checkbox"/>	Z221 <input type="checkbox"/>	Z231 <input type="checkbox"/>
* Fabric cover	CHOOSE Blue	Z244 <input type="checkbox"/>	Z246 <input type="checkbox"/>	Z248 <input type="checkbox"/>
	Pink	Z245 <input type="checkbox"/>	Z247 <input type="checkbox"/>	Z249 <input type="checkbox"/>
* Chest strap	CHOOSE With lateral positioning	Z212 <input type="checkbox"/>	Z222 <input type="checkbox"/>	Z232 <input type="checkbox"/>
	Without lateral positioning	Z213 <input type="checkbox"/>	Z223 <input type="checkbox"/>	Z233 <input type="checkbox"/>
Head blocks		Z214 <input type="checkbox"/>	Z214 <input type="checkbox"/>	Z234 <input type="checkbox"/>
Leg straps		Z215 <input type="checkbox"/>	Z225 <input type="checkbox"/>	Z235 <input type="checkbox"/>
Calf rest (Calf rest must be chosen at the time of the original purchase. It cannot be added later.)		Z216 <input type="checkbox"/>	Z226 <input type="checkbox"/>	Z236 <input type="checkbox"/>
Tub stand		Z217 <input type="checkbox"/>	Z217 <input type="checkbox"/>	Z217 <input type="checkbox"/>
Shower stand		Z218 <input type="checkbox"/>	Z218 <input type="checkbox"/>	Z218 <input type="checkbox"/>
Tub transfer base (calf rest recommended)	CHOOSE Counterclockwise into tub	Z227 <input type="checkbox"/>	Z227 <input type="checkbox"/>	Z227 <input type="checkbox"/>
	Clockwise into tub	Z228 <input type="checkbox"/>	Z228 <input type="checkbox"/>	Z228 <input type="checkbox"/>
Legs for tub transfer base	CHOOSE Standard	Z224 <input type="checkbox"/>	Z224 <input type="checkbox"/>	Z224 <input type="checkbox"/>
	Long (tub edge over 51 cm high)	Z238 <input type="checkbox"/>	Z238 <input type="checkbox"/>	Z238 <input type="checkbox"/>
Additional head blocks		Z214 <input type="checkbox"/>	Z214 <input type="checkbox"/>	Z234 <input type="checkbox"/>
Additional fabric cover		Z219 <input type="checkbox"/>	Z229 <input type="checkbox"/>	Z239 <input type="checkbox"/>
Additional chest strap	CHOOSE With lateral positioning	Z212 <input type="checkbox"/>	Z222 <input type="checkbox"/>	Z232 <input type="checkbox"/>
	Without lateral positioning	Z213 <input type="checkbox"/>	Z223 <input type="checkbox"/>	Z233 <input type="checkbox"/>
Total		Z210	Z220	Z230

* Required